## **HATHO ThermoPol Set**

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## **HATHO ThermoPol Set**

Ultrafast polishing for brilliant shine:  
3 steps, 3 benefits, 3 winners

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Save time, increase brightness: The new HATHO ThermoPol Set improves the surface of thermoplastic dentures quicker than ever before.

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**The secret  
of pre-polishing**

- ★ Attach the HATHO Multi Layer Brush art. no. 4280 to your polishing lathe.
- ★ Adjust the speed to low speed, at 1,800 rpm.
- ★ To fill the multi layer brush with additional polishing agent, hold the pre-polishing compound HATHO Polistar Lintygrey ref. no. 313168 against the rotating multi layer brush for approx. 2 seconds.

- ★ Use the filled brush to evenly polish the thermoplastic denture surfaces for approx. 3 to 4 minutes until smooth.
- ★ Finish step 1 as usual by cleaning the workpiece.

- 2 x Art. Nr. 4280: HATHO Einlagebürste ø 80 mm
- 2 x art. no. 4280:
- 3" HATHO Multi Layer Brush
- 1 x Ref. Nr. 313168: HATHO Polistar Lintygrau 90 g
- 1 x ref. no. 313168:
- HATHO Polistar Lintygrey 90 g
- 1 x Art. Nr. 869 100/14: HATHO COSIMA Soft Polierschwabbel ø 100 mm
- 1 x art. no. 869 100/14:
- 4" HATHO COSIMA Soft Polishing Buff
- 1 x Ref. Nr. 313200: HATHO Polistar Creme 90 g
- 1 x ref. no. 313200:
- HATHO Polistar Cream 90 g
- 1 x Art. Nr. 868 4" x 50 H: HATHO MIRA Polierschwabbel ø 100 mm
- 1 x art. no. 868 4" x 50 H:
- 4" HATHO MIRA Polishing Buff
- 1 x Ref. Nr. 313196: HATHO Polistar Pro Rosa 150 g
- 1 x ref. no. 313196:
- HATHO Polistar Pro Pink 150 g

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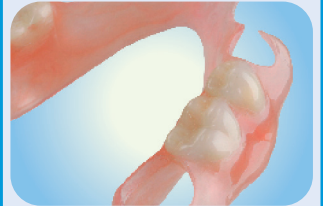
**The technique  
of shine polishing**

- ★ Attach the HATHO COSIMA Soft Polishing Buff art. no. 869 100/14 to your polishing lathe.
- ★ Adjust the speed to low speed, at 1,800 rpm.
- ★ Apply approx. 0.5" of HATHO Polistar Cream ref. no. 313200 to the pre-polished thermoplastic dentures as needed.

- ★ Gently and evenly polish the denture surfaces for approx. 3 to 4 minutes until all fine marks have disappeared.
- ★ Finish step 2 as usual by cleaning the workpiece.

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Everyone benefits from the perfect result: Dental technicians, dentists and patients alike.

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**The discovery  
of high shine polishing**

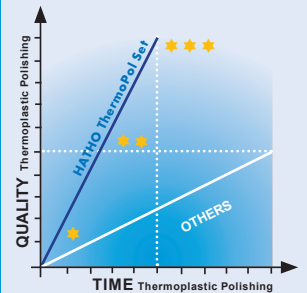
- ★ Attach the HATHO MIRA Polishing Buff art. no. 868 4" x 50 H to your polishing lathe.
- ★ Adjust the speed to low speed,

at 1,800 rpm.

- ★ Fill the polishing buff with polishing agent by holding high shine polishing compound HATHO Polistar Pro Pink ref. no. 313196 against the rotating buff for approx. 2 seconds.
- ★ Gently and evenly polish the denture surfaces for approx. 3 to 4 minutes until they exude the high shine, brilliant appearance that up until now could only be achieved with PMMA acrylic dentures!
- ★ Finish step 3 as usual by cleaning the workpiece.

- ★ Enjoy the best result in polishing thermoplastic dentures achieved in the shortest time ever needed.

**Blitzschnell, blitzblank**  
High shine in record time

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Outstanding advantages in polishing of thermoplastic dentures: **Maximum brightness in minimum time.**

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Powerful abrasives prepare the way

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HATHO Art. Nr. 4280

3" HATHO Multi Layer Brush with 2 outer rows of white Chungking bristles and 2 inner layers of Scotch Brite™, as well as HATHO UniFix plastic core for better fixing on the tapered spindle. Suitable for up to 3,500 rpm.

Professional pre-polish brush for pre-polishing acrylic dentures (PMMA), thermoplastics (Nylon), 3D-acrylics (rapid prototyping) and light curing impression trays.

Part of the new HATHO ThermoPol Set. Also available separately. 12 units per packaging unit.

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90 g HATHO Polistar Lintygrey polishing compound made from soft wax mixture with fine grit.

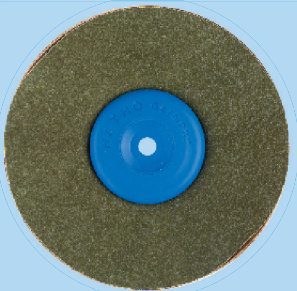
Professional polishing agent for pre-polishing acrylic dentures (PMMA), thermoplastics (Nylon), 3D-acrylics and light curing impression trays.

Part of the new HATHO ThermoPol Set. Also available separately. Units à 90 g, 300 g or 1,2 kg.

HATHO ref. no. 313168 (90 g)  
HATHO ref. no. 313175 (300 g)  
HATHO ref. no. 313150 (1,2 kg)

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Soft buff achieves smooth surfaces

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HATHO Art. Nr. 869 100/14

4" HATHO COSIMA Soft Polishing Buff made out of 14 layers of fine, siliconized cotton cloth positioned on a plastic core. Suitable for up to 3,500 rpm.

Professional polishing buff for shine polishing acrylic dentures (PMMA), thermoplastics (Nylon), 3D-acrylics, light curing impression trays, titanium, precious and non-precious metal alloys.

Part of the new HATHO ThermoPol Set. Also available separately. 1 unit per packaging unit.

HATHO art. no. 869 100/14

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Polishing cream gives sparkling brilliance

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HATHO Ref. Nr. 313200

90 g HATHO Polistar Cream highly filled with fine grit.

Professional shine polishing cream for shine polishing acrylic dentures (PMMA), thermoplastics (Nylon), 3D-acrylics, light curing impression trays, precious metal alloys.

Free from surfactants and solvents.

Part of the new HATHO ThermoPol Set. Also available separately. 1 unit per packaging unit.

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HATHO Art. Nr. 868 4" x 50 H

4" HATHO MIRA Polishing Buff made out of 50 layers of the finest natural cotton cloth, not sewn. Suitable for up to 3,500 rpm.

Professional polishing buff for the high shine polishing of acrylic dentures (PMMA), thermoplastics (Nylon), 3D-acrylics, light curing impression trays, frame works, titanium, precious and non-precious metal alloys.

Part of the new HATHO ThermoPol Set. Also available separately. 1 or 10 units per packaging unit.

HATHO art. no. 868 4" x 50 H

## Die Beste für's Finish

Ultimate polishing creates flawless finish

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HATHO Ref. Nr. 313196

HATHO Polistar Pro Pink polishing compound. 150 g handcrafted high shine polishing compound with ultra fine grit.

Professional polishing compound for the high shine polishing of acrylic dentures (PMMA), thermoplastics (Nylon), 3D-acrylics and light curing impression trays.

Part of the new HATHO ThermoPol Set. Also available separately. 1 unit per packaging unit.

HATHO ref. no. 313196